## FR Thermomètre digital flexible

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## EN Flexible Digital Thermometer

Please read the instructions carefully before using the thermometer for the first time, and keep these in a safe place. This product is intended for the measurement of human body temperature.

### OPERATING INSTRUCTIONS

Before using, please disinct the probe first. To switch on, press the ON/OFF button next to the display, a short beep will sound, indicating that the thermometer is now operational. At the same time the thermometer runs a self-check test, during which all the digital segments appear on the LCD. When the letters "Lo" and a flashing "C" display, the thermometer is now ready for use. If the ambient temperature is below 32°C or 89,6°F, then "Lo" will appear on the LCD and it is above 42°C / 107,6°F, then "Hi" will appear on the LCD.

During the reading, the current temperature is displayed constantly and the "C" symbol flashes. As soon as the constant temperature value is reached, a beep will sound four times, and the "C" symbol will stop flashing. The highest temperature measured appears on the LCD. However, please note that this thermometer is a maximum thermometer, i.e. the displayed temperature can increase slightly if measurement continues after the beep. This is particularly the case with axillary temperature, where a temperature value must be recorded which approximates the core body temperature.

In this instance please note the description under "Methods of measuring temperature". When the measurement is completed, please switch the thermometer off by pressing the ON/OFF button. After the temperature has been displayed, the thermometer will shut off automatically after 5 minutes from the beep.

Note: We strongly recommend the rectal method as the most accurate method for identifying the basal temperature and advise you to extend the measuring time by 3 minutes after the beep.

### CLEANING AND DISINFECTION

The best way to clean the thermometer tip is by applying a disinfectant (e.g. 70% ethyl alcohol) with a damp cloth. The thermometer is waterproof and can be immersed in liquid or lukewarm water for thorough cleaning and disinfection.

### SAFETY FUNCTIONS

• Do not allow the device to come into contact with hot water.

• Do not expose to high temperatures or direct sunlight.

• Do not drop the thermometer. It is neither shock-proof nor impact-resistant.

• Do not modify this device without authorisation of manufacturer.

• Do not clean with thinners, petrol or benzene. Only clean with water or disinfectant.

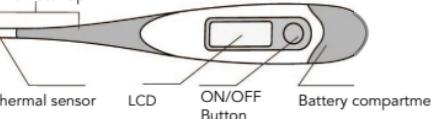
• Do not immerse the waterproof thermometers under water 15cm for longer than 30 minutes.

• Do not bend or open the device (except the battery compartment).

• The thermometer contains small parts (battery, battery compartment) which can be swallowed by children. For this reason, do not leave the thermometer unattended in the hands of children.

The usual measuring time is approximately 40 to 60 seconds.

#### Thermometer tip

Thermal sensor

LCD

ON/OFF Button

Battery compartment

**LOT** Lot number

**REF** Reference

**CE 0197** CE conformity marking

**Common signage TRIMAN**

### CALIBRATION CHECK

This thermometer is initially calibrated at the time of manufacture.

### WARRANTY

This product is warranty for 2 year from the date of purchase. Damage resulting from incorrect use or abuse is not covered by the warranty. Battery and packaging are excluded from the warranty. Claims beyond this, including claims for damages, are excluded. If you find that the thermometer is defective and not in good function, please firstly check the battery before sending in for repair.

#### TECHNICAL DATA

• Type: maximum thermometer

• Measurement range: (32.0 – 42.0)°C

• Measurement accuracy:

+/- 0.1°C (35.5°C – 42.0°C)

+/- 0.2°C (32.0°C – 35.5°C)

• Storage/transportation temperature:

-20°C – +55°C, -95% RH

• Ambient temperature during use: (5-35)°C, <80% RH

• Min Scale: 0.1°C

• Battery type: Alkaline manganese battery, type LR41, 1.5V

• Service life minimum 100 hour under continuous operation.

• Weight Approx. 12g

### EXPLICATION DES SYMBOLES

battery is empty

Product disposal instructions for electronic device

The battery in this product complies with the requirements in European Directives 2006/66/EC

Temperature under 32°C

Temperature over 42°C

Type BF equipment

Read IUD carefully

Classification according to the degree of protection against ingress of water as detailed in IEC 60529

Storage condition

Manufactured by

European Authorized Representative

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Shanghai International Holding Corp., GmbH
Hangzhou Hu'an Medical & Health Instruments Co., Ltd.