**PRINCIPE**

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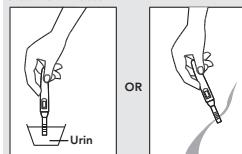
For self-testing.

**INSTRUCTIONS**

1. Take the test out of its aluminum pouch and use it within one hour.
2. Remove the cap from the pregnancy test and direct your urine stream onto the strip, or submerge it for at least 15 seconds in a urine sample collected into a clean container. Ensure that the 5 holes in the plastic part of the test are placed in contact with the urine.
3. Replace the cap onto the tip, place the product on a clean and stable surface and start the stopwatch immediately.
4. The result should be read after three minutes at the least and will no longer be reliable after 10 minutes.

**READING THE RESULTS****POSITIVE**

Two coloured lines will appear. One of the lines will be in the control area (C) and the other in the test area (T). The fact that one line is lighter than the other has no impact on the results.

This means that you are probably pregnant.

**NEGATIVE**

A coloured line will appear in the control area (C).

No line will appear in the test area (T).

This means that you are probably not pregnant.

**INVALID**

The result is invalid when no coloured line appears in the control area (C), even if a line appears in the test area (T). Repeat the procedure with a new pregnancy test.

**RESTRICTIONS**

1. Medicinal products that contain hCG (such as Pregnyl, Profasi, Pergonal and APL) can give false positive results. Alcohol, oral contraceptives, painkillers, antibiotics and hormone treatments that do not contain hCG should not affect the test result.
2. Apart from pregnancy, several medical conditions such as ovarian cysts or an extrauterine pregnancy (developed outside the uterus) may lead to high hCG levels and give a false positive result.
3. This test provides a presumed pregnancy diagnosis. A confirmed pregnancy diagnosis can only be made by a doctor after assessing clinical and laboratory results.

**ADDITIONAL INFORMATION****1. How does the pregnancy test work?**

The Simply pregnancy test detects a hormone that your body produces during pregnancy (human chorionic gonadotropin - hCG) in your urine. The hCG hormone level increases the further along in your pregnancy you are.

**2. After how long should I use the test if I suspect I might be pregnant?**

You can use the Simply pregnancy test the day after the expected date of your period. You can perform the test at any time of day. However, if you are pregnant, be aware that first-morning urine has the highest concentration of the hCG hormone.

**3. Do I have to take the test using first-morning urine?**

Although you can perform the test at any time of day, first-morning urine is usually the most concentrated of the day and therefore contains the most hCG.

**4. How accurate is the test?**

A comparative clinical study was performed on the Simply pregnancy test and a reference urine hCG test. The reference test is a control test that makes it possible to assess the reliability, accuracy and sensitivity of a urinary pregnancy test. The Simply pregnancy test and the reference test identified the same results across 413 samples; the tests identified 117 positive results and 296 negative results. Compared to the reference test, the overall reliability of the Simply pregnancy test was higher than 99%.

**5. How sensitive is the test?**

The Simply sensitive pregnancy test detects hCG in urine at a concentration equal to or higher than 25 mIU/ml. The test complies with the WHO international standard. Adding LH (300 mIU/ml), FSH (1,000 mIU/ml) and TSH (1,000 µU/ml) to negative (0 mIU/ml hCG) and positive (25 mIU/ml hCG) samples did not lead to any cross reactions.

**6. What should I do if the result is positive?**

This means that your urine contains hCG and that you are probably pregnant. Consult your doctor to confirm your pregnancy and together discuss all the steps to take.

**7. How can I be sure that I performed the test correctly?**

A coloured line will appear in the control area (C), which indicates that you have followed the test procedure correctly and that a sufficient amount of urine has been absorbed.

**8. What should I do if the result is negative?**

This means that the hCG hormone was not detected in your urine and that you are probably not pregnant. If your period does not occur or stop after its expected start date, repeat the procedure with a new pregnancy test. If the result is negative again after this new test and your period is still late, consult your doctor.

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## BIBLIOGRAPHIE / BIBLIOGRAPHY / BIBLIOGRAFIE / LITERATUR / BIBLIOGRAFIA

- Steier JA, P Bergsjo, OL Myking Human chorionic gonadotropin in maternal plasma after induced abortion, spontaneous abortion and removed ectopic pregnancy. Obstet. Gynecol. 1984; 64(3): 391-394.
- Dawood MY, BB Saxena, R Landesman Human chorionic gonadotropin and its subunits in hydatidiform mole and choriocarcinoma, Obstet. Gynecol. 1977; 50(2): 172-181.
- Braunstein GD, JL Vaitukatis, PP Carbone, GT Ross "Ectopic production of human chorionic gonadotropin by neoplasms", Ann. Intern. Med. 1973; 78(1): 39-45.

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