PRINCIPE DU TEST

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TEST PRINCIPLE

The **self-test for VAGINAL INFECTIONS** is a reliable, painless method to determine vaginal pH. As soon as the pH measurement area of the applicator comes into contact with vaginal secretions, its colour changes. The colour obtained corresponds to a pH value on the measuring scale and gives the test result. Regular use of the self-test for vaginal infections test is a reliable way to detect vaginal flora disorders.

INFORMATION

An acid vaginal pH (between 3.8 and 4.4) is essential for the proper functioning of the vaginal protection system. Contamination by pathogenic germs and the occurrence of vaginal infections can therefore be effectively prevented. Many factors regulate the composition and functioning of the vaginal microbial ecosystem and preserve its natural protection function. Lactobacilli are one of the main factors. The most important and most natural protection against vaginal disorders is therefore a healthy vaginal flora.

We recommend that you perform the test twice a week. By doing so, you can ensure that your vaginal pH is correct between two visits to your gynaecologist.

Bacterial vaginosis is diagnosed in around 15 to 20% of pregnant women.

By measuring their vaginal pH regularly, pregnant women can reduce the risk of infection.

Vaginal pH can increase in the event of:

- Bacterial vaginosis
- Sexually transmitted diseases
- Premature rupture of the amniotic sac (waters breaking)

STORAGE

Keep the test at room temperature in the closed cardboard box, in a place out of reach of children. If you have any questions about this product, please call Customer Service in your country. Our staff are at your disposal.


BOX CONTENTS

- 3 test applicators packed in aluminum pouches
- 1 instruction leaflet
- 1 colour scale printed on each aluminum pouch

READ OF THE RESULT

Read the result as soon as the applicator is removed from the vagina, while the measuring area is still wet. Compare the colour of the pH measurement area with the colour scale and select the colour closest to your measurement.

After use, dispose of the applicator in household waste.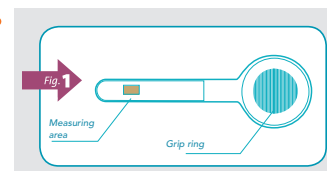

PRECAUTIONS FOR USE

Use each applicator only once.

Use this product only for its intended purpose. Do not swallow.

The applicator only measures vaginal pH and does not detect infections.

An acid pH is not always enough to prevent infections. If you notice any disorders despite a normal pH value, consult your doctor.

Use the test before the expiry date (see date on the box).

Certain circumstances may temporarily change vaginal pH and thereby falsify the results of this measurement. Take the following limitations into account when measuring your pH and interpreting the test.

Perform the measurement:

- at least 12 hours after sexual intercourse.
- at least 12 hours after vaginal administration of medicinal products (pessaries, cream, gel, etc.).
- 3 to 4 days after your period ends.
- at least 15 minutes after urinating: traces of urine may lead to a false result.

Do not use after washing or showering external genital areas.

Never start a treatment without discussing the test result with your doctor.

Incorrect use of the applicator may cause deflowering in virgin women.


WHEN TO PERFORM THE TEST?

- In the event of bacterial vaginosis
- After anti-infective treatment (e.g. antibiotics)
- As part of preventive examinations
- During pregnancy
- In case of a history of premature births
- After sexual intercourse with a new partner
- In the event of discomfort in the genital organs (burning, itching, unpleasant smell, discharge)
- Before and after surgery in the vagina
- If there is a history of premature delivery
- After sexual intercourse with a new partner
- In case of discomfort in the genitals (burns, pruritus, unpleasant odours, losses)
- Before and after vaginal surgery

DIRECTIONS FOR USE

Measuring vaginal pH is easy and completely painless. Relax and follow the instructions below very precisely.

1. Wash your hands before starting the test.
2. Open the box, take out a test and open the plastic pouch.
3. Hold the applicator by the grip ring between the thumb and index finger, as shown in the illustration below (Figure 1). The thumb must always be on the side of the vaginal applicator, as shown.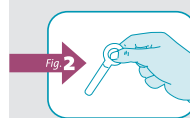


4. Avoid the pH measuring area coming into contact with other liquids.


5. Sit down with the knees apart or stand with one foot on the toilet seat or another stable surface.

6. With your free hand, gently free the opening of the vagina by spreading the external labia.

7. Gently introduce the applicator into the vagina as if you were using a tampon (Figure 2) up to about 1 to 3 cm deep (no further than the grip ring allows). Inserting the vaginal applicator is completely painless.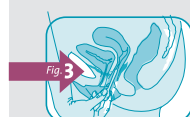

8. Press the tube gently against the posterior wall of the vagina (on the side of the rectum) for about 10 seconds to ensure that the pH measuring area is well moistened. Sufficient moistening is very important for the test's interpretation.
9. Finally, carefully remove the applicator from the vagina.

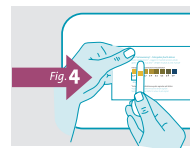For the interpretation of the test, always use the color scale printed on the packaging.

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INTERPRETATION OF THE RESULT

Normal vaginal pH (3.8 - 4.4)

You probably have thrush if you also have the following genital disorders:

- Itching
- Burning
- Abnormal discharge resembling "curdled milk"

Consult your pharmacist, who can recommend a suitable treatment. If the problems persist, consult a doctor.

Abnormally high vaginal pH (> 4.4)

You probably have a bacterial infection (vaginosis) if you also have the following genital disorders:

- Itching
- Unpleasant, fishy smell
- Abnormal liquid white to greyish discharge

You probably have a Trichomonas if you also have the following genital disorders:

- Pain when urinating
- Unpleasant mould-like smell
- Abnormal foamy yellow to greenish discharge

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