BIO SYNEX

ThermoFlash®

LX-260TE

VISIONED TECHNOLOGY CO., LTD.

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LX-260TE

USER MANUAL

CLASSIFICATION: CLASS IIA MEDICAL DEVICE.

THERMOFLASH® LX-260T EVOLUTION has been clinically tested and is approved according to the standard ASTM 1965-1998 (2009).

CONTENTS

I. SAFETY INSTRUCTIONS.....	14
II. INTRODUCTION	15
III. PRECAUTIONS BEFORE USE	15
IV. OPERATING PRINCIPLE	16
V. FUNCTIONS	17
VI. MODALITIES AND START-UP OF THERMOFLASH® LX-260TE.....	18
VII. MENU SETTING & FUNCTIONS	18
VIII. PRECISION.....	19
IX. ACCESSORIES SUPPLIED	19
X. TROUBLESHOOTING	19

I. SAFETY INSTRUCTIONS

- Follow the maintenance instructions in this manual.
- This device is suitable for personal home use.
- This device must only be used for the purpose described in this notice.
- This device must only be used in an ambient temperature range between 10°C and 40°C / 50°F and 104°F.
- This device must always be stored in a clean, dry area.
- Do not expose this device to electric shock.
- Do not expose this device to extreme temperatures $T^{\circ} > 50^{\circ}\text{C} / 122^{\circ}\text{F}$ - $T^{\circ} < -20^{\circ}\text{C} / -4^{\circ}\text{F}$.
- Do not use this device in relative humidity greater than 85%.
- Do not expose this device to direct sunlight or water.
- Do not use this device outside.
- Never drop the device. A shock may damage it.
- Do not attempt to service this device yourself.
- Do not attempt to open the device. In case of problems, contact the after sales service department.
- Do not leave this device within reach of children.
- This electronic medical device requires specific precautions regarding electromagnetic compatibility. It must be installed and commissioned in accordance with the electromagnetic

information provided in the Electromagnetic compatibility (EMC) Data table (page 53).

- Measurements may be impaired if used near a TV, microwave oven, mobile phone or other devices with an electric field.
- The measurement results are given for information only. If unsure about your results, please contact your doctor.
- This device should not encourage self-medication or adjusting your treatment.
- Stop using the device in the event of an abnormality or malfunction.
- In general, portable and mobile RF communication equipment does not affect the operation of this medical device. However, it cannot be used in an environment with high electromagnetic interference.

The manufacturer reserves the right to modify the product technical specifications without prior notice.

II. INTRODUCTION

The **THERMOFLASH® LX-260T EVOLUTION** is a revolutionary electronic medical infrared thermometer, equipped with the all-new automatic calibration technology MicroSecondFlash™ developed by **Visiomed®**. Easy to use, it measures the body temperature at a distance of 2 to 5 cm (0.79 to 1.97 in), in less than one second. **ThermoFlash®** is an infrared thermometer specially designed to take the body temperature of children or adults without contact with the body. It can be used at home or by health professionals.

However, it is imperative to use the **ThermoFlash®** appropriately in order to obtain a reliable and stable result. It is therefore advisable to read the instruction manual and safety instructions before use. **ThermoFlash®** is suitable for the whole family.

AGE	°C	°F
0-2	36.4 - 38.0	97.5 - 100.4
3-10	36.1 - 37.8	97.0 - 100
11-65	35.9 - 37.6	96.6 - 99.7
> 65	35.8 - 37.5	96.4 - 99.5

III. PRECAUTIONS BEFORE USE

The **THERMOFLASH® LX-260T EVOLUTION** is preset at the factory. It is not necessary to calibrate the device when starting it up.

In order to obtain a reliable and stable result, it is advisable at every change of environment, with significant ambient temperature difference, to leave the **THERMOFLASH® LX-260T EVOLUTION** at this ambient temperature for 15 to 20 minutes before using it. Before any new temperature measurement, wait for the screen to turn off. It is important to respect a minimum of 3 minutes between each temperature measure.

- Clean the glass with a cotton swab slightly soaked in 70° alcohol.
- Do not put the thermometer in contact with diseased, ulcerated or wounded skin.

- Do not drink hot or cold drinks, and do not do strenuous exercise during temperature taking.
- The protective glass of the lens is the most important and fragile part of the thermometer, treat it carefully and do not touch it with your fingers.

Constraints

In order to obtain a stable and reliable result, before taking the temperature:

- Move hair aside.
- If necessary, wipe away any perspiration from the forehead.
- Avoid drafts/any significant air flow (nasal cannula, air conditioning ...).
- Observe a 3-minute interval between two measures.

The reliability of the measure is not assured in case of temperature measurement on another anatomical region (e.g. arms, torso ...).

IV. OPERATING PRINCIPLE

Each solid, liquid or gaseous body, has the property of emitting energy by radiation with an intensity that depends on its temperature. The **THERMOFLASH® LX-260T EVOLUTION** infrared thermometer measures a person's temperature by means of the energy they emit. This measurement is made possible by the instrument's external temperature probe, which permanently analyses and records the ambient temperature. In such a way that as soon as the operator holds the thermometer close to the body and activates the radiation sensor, the measurement is carried out instantaneously by detection of the infrared heat emitted by the arterial blood flow. The result of this body temperature measurement is then obtained without interference from the heat of the surrounding environment.

INFORMATIVE TABLES

Normal temperatures according to procedure

MEASUREMENT MODE	NORMAL T°
Rectal	36.6°C – 38°C / 97.88°F – 100.4°F
Oral	35.5°C – 37.5°C / 95.9°F – 99.5°F
Axillary	34.7°C – 37.3°C / 94.46°F – 99.14°F
Aural	35.8°C – 38°C / 96.44°F – 100.4°F
Temporal (Thermoflash®)	35.8°C – 37.8°C / 96.44°F – 100.04°F

Chamberlain, J.M., Terndrup, T.E., *New Light on Ear Thermometer - Readings, Contemporary Pediatrics*, March 1994.

Chamberlain, J.M., et al., *Determination of Normal Ear - Temperature with an Infrared Emission Detection Thermometer*

Advantages of temporal artery (TA) temperature

The infrared arterial temperature can be measured using a device placed on the forehead in the temporal artery region. The **THERMOFLASH® LX-260T EVOLUTION** thermometer is designed for instant temple/forehead temperature measurement without physical contact with the temporal artery. This artery being relatively close to the surface of the skin and therefore accessible, the blood flow is permanent and regular and allows an accurate measurement of the temperature. This artery is linked to the heart by the carotid artery, which is itself directly linked to the aorta. It is part of the central arterial system. The efficiency, speed and comfort of taking a temperature from this area make it ideal.

Practical considerations when taking a temperature

- In the interests of precision and measurement quality, it is essential that each user is adequately informed and trained in the temperature measurement technique when using such a device.
- It is essential to remember that the procedure of taking a temperature although simple, should not be ignored.
- The patient should not carry out strenuous activities before taking their temperature.
- Temperature fluctuations due to physiological factors must be taken into consideration in the evaluation of the results:
 - The temperature increases by 0.5°C/0.9°F between 6:00 and 15:00,
 - The average temperature of women is about 0.2°C / 0.36°F and varies according to their ovarian cycle,
 - It increases by 0.5°C/0.9°F during the first half of the cycle and during the first months of pregnancy.
 - Age, sex, type and thickness of skin.
- The temperature of a seated person is lower by around 0.3°C - 0.4°C/0.54°F - 0.52°F than that of a person standing.

*Haut Conseil de Santé Publique [online]. Health Recommendations of the National Heatwave Plan 2014. P123 [Consulted on 25 July 2016]
Available at http://social-sante.gouv.fr/IMG/pdf/Recommandations_sanitaires_Plan_canicule_2014.pdf*

V. FUNCTIONS

1. A reliable and stable measurement result, thanks to the HEIMMANN infrared detection system.
2. Audible alarm if the temperature exceeds the predefined threshold.
3. Memorisation of the last 32 temperature measurements.
4. Data displayed in Celsius or Fahrenheit.
5. Automatic stop (energy saver).
6. Vocal indication of measurements and color indicators to quickly display and interpret the diagnosis of the temperature obtained.

Additional uses

The **THERMOFLASH® LX-260T EVOLUTION** can also measure the temperature of a surface or room temperature (in Surface or Room mode).

VI. MODALITIES AND START-UP OF THERMOFLASH® LX-260T EVOLUTION

1. Install the batteries.
2. When the batteries are used for the first time or when inserting new batteries, wait about 15 minutes for the device to reach room temperature.
3. Aim the device at the temporal lobe or a side/temple, as shown (SEE DIAGRAM **A** PAGE 2), at a distance of 2 to 5 cm (0.79 - 1.97 in) press the measurement key, the temperature is instantly displayed. The temperature can also be taken behind the ear lobe - on the jugular vein.

VII. MENU SETTING & FUNCTIONS

1. Choice of temperature display mode

Body mode: Measurement of the body temperature.

Measuring range: 32°C to 42.9°C (89.6°F to 109.2°F)

Surface mode: Measurement of the surface temp of an object or food.

Measuring range: 0°C to 93.3°C (32.0°F to 199.94 °F)

Room mode: Measure the ambient temperature of a room.

Measuring range: 0°C to 50°C (32°F to 122°F)

Press the measurement button to turn on the device.

Press MODE button to select the temperature measurement mode.

The screen displays: Body mode - for body temperature measurement. Press the MODE button again to switch to SURFACE mode. The screen displays Surface.

Press the MODE button again to switch to ROOM mode. The screen displays Room.

Note: the THERMOFLASH LX-260T EVOLUTION is automatically set to BODY.

IMPORTANT: The area temperature differs from the internal body temperature.

Use BODY mode only to obtain the internal body temperature.

2. Choice of temperature unit

Press the °C/°F button to select the temperature unit. The screen, displays the symbol of the selected unit (°C for degrees Celsius, °F for degrees Fahrenheit).

3. Data memory

Press MEM for two seconds to display the last measure. Press MODE and C°/F° to scroll through the last recorded measurements. To exit the menu, press the measure key. The device automatically switches off after 5 seconds of inactivity.

4. Choosing the language

Use the Language key to choose a language for the vocal function. The choice of language is visible.

Languages available: French, English, Spanish, Italian, German, Dutch, Silent Function. By default the ThermoFlash® LX-260T EVOLUTION is in French.

5. Alarm threshold

The alarm threshold is set to 38°C. This cannot be changed. If the temperature is over 38°C, a beep sounds during the temperature measurement.

6. Changing the batteries

When the LCD screen displays , the battery is used. Operation: Open the cover and replace the used batteries taking care to observe polarity. A polarity error may cause damage and void the guarantee. Do not use batteries other than the batteries mentioned, do not recharge non-rechargeable batteries, or throw them into a fire. Remove the batteries when not using the device for a long period. Use disposable batteries.

VIII. PRECISION

ACCORDING TO ASTM STANDARD 1965-1998 (2009)	34°C - 35.9°C = ± 0.3°C	93.2°F - 96.6°F = ± 0.3°F
	36°C - 39°C = ± 0.2°C	96.8°F - 102.2°F = ± 0.2°F
	39°C - 42.5°C = ± 0.3°C	102.2°F - 108.5°F = ± 0.3°F

The **THERMOFLASH® LX-260TE** can take temperature measurements below 32°C / 89.6°F or above 42.9°C / 109.22°F. Outside these temperature ranges, precision is not guaranteed.

Longevity of use

The **THERMOFLASH LX-260T EVOLUTION** is guaranteed for about 40,000 readings.

IX. ACCESSORIES SUPPLIED

User manual, Carrying bag, Batteries supplied LR6 (AA).

X. TROUBLESHOOTING

If you experience any of these problems using your **THERMOFLASH® LX-260T EVOLUTION**, please refer to this troubleshooting guide to resolve the problem. If the problem persists, please contact our customer service at +33 892 350 334 (0.40€ per minute).

- **The display indicates temperatures above 95°:** the temperature is indicated in Fahrenheit. Change the unit of measure to Celsius.
- **The screen displays temperatures below 32°C (89.6°F) :** to measure body temperature, the mode BODY must be selected. If the thermometer is in SURFACE or ROOM mode, the displayed reading indicates the external temperature released by your body.
- **The screen displays the message HI :** the analysed temperature is above the range of the selected mode, i.e. greater than 42.9°C (109.2°F) in BODY mode, or 60°C (140°F) in SURFACE mode.
- **The screen displays the message LO :** The analysed temperature is below the measuring range of the selected mode, i.e. less than 32°C (89.6°F) in BODY mode and less than 0°C (32°F) in SURFACE mode.

This message displays in various cases - below are a list of the main cases:

CAUSES OF LO MESSAGE DISPLAY	SOLUTIONS
Temperature reading hampered by hair, sweat,...	Make sure there is no obstruction prior to taking the temperature.
Temperature reading hampered by an air flow...	Be careful not to be in the axis of an airflow, this could cause interference with the infrared system.
Temperature reading too close together, ThermoFlash® has not had time to reset.	Please observe the minimum interval of 1 minute between two readings - a 3 min delay is advised.
The measuring distance is too great.	Please respect the measuring distance (between 2 and 5 cm / 0.79 - 1.97 in).

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EMC DATA

EMC data

Guidance and manufacturer's declaration – electromagnetic emission - for all EQUIPMENT AND SYSTEMS

The model **LX260TE** is intended for use in the electromagnetic environments specified below. the customer or the user of the model **LX-260TE** should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Model LX-260TE uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The Model LX-260TE is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	N/A	
Voltage fluctuations /flicker emissions IEC 61000-3-3	N/A	

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	± 8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines 100 kHz repetition frequency ±1 kV for input/output lines	N/A	N/A
Surge IEC 61000-4-5	± 0.5 kV, ± 1 kV differential mode line-line	N/A	N/A

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT (100 % dip in UT) for 0.5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270°, and 315° 0 % UT (100 % dip in UT) for 1 cycle at 0° 70 % UT (30 % dip in UT) for 25/30 cycles at 0° 0 % UT (100 % dip in UT) for 250/300 cycle at 0°	N/A	N/A
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m, 50/60Hz	30 A/m, 50/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Note: *UT is the a. c. mains voltage prior to application of the test level.*

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 6 Vrms 150 kHz to 80 MHz outside ISM bands	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the Models LX-260TE , including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$ 80MHz to 800MHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800MHz to 2.7GHz
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz	10 V/m	where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres(m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range ^b Interference may occur in the vicinity of equipment marked with the following symbol:

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a. The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

b. The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2,7 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an

additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in these frequency ranges.

c. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the **LX-260TE** is used exceeds the applicable RF compliance level above, the **LX-260TE** should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the **LX-260TE**.

d. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM - Recommended separation distances between portable and mobile RF communications equipment and the model LX-260TE.

The Model **LX-260TE** is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Model **LX-260TE** can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Model **LX-260TE** as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	80 kHz to 800 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.7 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Recommended separation distances between RF wireless communications equipment.

The device is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the device can help prevent electromagnetic interference by maintaining a minimum distance between RF wireless communications equipment and the device as recommended below, according to the maximum output power of the communications equipment.

Frequency MHz	Maximum Power W	Distance	IEC 60601 Test Level	Compliance Level
385	1.8	0.3	27	27
450	2	0.3	28	28
710	0.2	0.3	9	9
745				
780				
810	2	0.3	28	28
870				
930				
1720	2	0.3	28	28
1845				
1970				
2450	2	0.3	28	28
5240	0.2	0.3	9	9
5500				
5785				

Electromagnetic Environment - Guidance

RF wireless communications equipment should be used no closer to any part of the device, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

Recommended separation distance $E = [\frac{6}{d}] \sqrt{P}$

Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitter, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol: 

Note 1: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

WARNING

- *This device should not be used in the vicinity or on the top of other electronic equipment such as cell phone, transceiver or radio control products. If you have to do so, the device should be observed to verify normal operation.*
- *The use of accessories and power cord other than those specified, with the exception of cables sold by the manufacturer of the equipment or system as replacement parts for internal components, may result in increased emissions or decreased immunity of the equipment or system.*

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EN: BIOSYNEX will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

DURATION: 24 MONTHS RETURN TO WORKSHOP

LIMITS AND EXCLUSIONS: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by BIOSYNEX, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Visiomed* and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Visiomed*, any other guarantee (commercial guarantee) except this one will not be taken into account.

IMPORTANT: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the BIOSYNEX customer service.

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EN: *Disposal of Waste Equipment by Users in Private Households in the European Union.*

The symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. Instead, it is your responsibility to dispose of your waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. The separate collection and recycling of your waste equipment at the time of disposal will help to conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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© 02/2019 LX-260TE
EN: TensioFlash is a brand registred by BIOSYNEX. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registred design. The logos and trademarks are the property of their respective owners.