# TensioFlash®

**LABORATOIRES VISIOMED**  
112, Avenue Kléber  
75116 PARIS - France  
[contact@visiomed-lab.fr](mailto:contact@visiomed-lab.fr)  
[www.visiomed-lab.com](http://www.visiomed-lab.com)  
SAV : 0 800 680 080

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KD-795

By Visiomed

Medical device in compliance with current European standards.

**Please read before use.**

## TABLE OF CONTENTS

1. WARNING.....	19
2. BLOOD PRESSURE.....	20
3. PRECAUTIONS BEFORE USE.....	22
4. FEATURES OF THE TENSIOFLASH® KD-795.....	22
5. MAIN DEVICE.....	22
6. CHANGING THE BATTERIES.....	23
7. TIME AND DATE SETUP.....	23
8. VOICE SETTING.....	24
9. SOME SUGGESTIONS BEFORE MEASUREMENT.....	24
10. HOW TO USE THE WRIST CUFF.....	25
11. HOW TO MEASURE BLOOD PRESSURE.....	25
12. MEMORY STORAGE AND RECALL.....	26
13. DEVICE MAINTENANCE.....	26
14. PREVENT A MALFUNCTION.....	27
15. TECHNICAL SPECIFICATIONS.....	27
16. TROUBLESHOOTING GUIDE.....	28

*The manufacturer reserves the right to modify the product technical specifications without prior notice.*

Dear client,

Thank you for purchasing the VISIOMED upper arm automatic blood pressure monitor. We recommend that you read these instructions carefully, in order to measure your blood pressure effectively.

## 1. WARNING


- Follow the maintenance instructions in this manual.
- This device is suitable for home use.
- Use this device only for its intended use as described in this notice.
- Use this device in an ambient temperature range between 5 and 40°C.
- This device must always be stored in a clean, dry area.
- Do not expose this device to electric shock.
- Do not expose this device to extreme temperature conditions > 55°C or < 20°C.
- Do not expose this device to direct sunlight or water
- Do not use this device outside.
- Never drop the device.
- Do not attempt to service this device yourself. - Do not attempt to open the device. In case of problems, contact your dealer.
- This device is not intended for use by persons (including children) with impaired physical, sensory or mental capabilities, or persons with no experience or knowledge of the device, unless they have undergone supervision or prior instructions concerning the use of this device by a person responsible for their safety.
- This electronic medical device requires specific precautions regarding electromagnetic compatibility. It must be installed and commissioned in accordance with the electromagnetic information provided in the Electromagnetic compatibility (EMC) Data table
- Measurements may be impaired if used near a TV, microwave oven, mobile phone or other devices with an electric field.
- The measurement results are given for information only. If unsure about your results, please contact your doctor.

- This device should not encourage self-medication or adjusting your treatment.
- Stop using the device in the event of an abnormality or malfunction.

## 2. BLOOD PRESSURE

Blood pressure varies considerably throughout the day and in hypertensive individuals these variations are even more pronounced. It is lower in the morning and increases in the afternoon and evening. It decreases during sleep. Blood pressure is lower in summer than in winter and increases substantially with age. Blood pressure also increases during exercise, intercourse, pregnancy, stress

...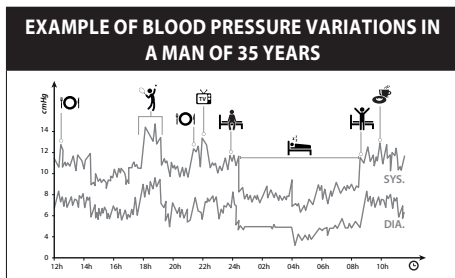

*Note: One or two readings will not provide a true indication of your normal blood pressure. It is important to take regular, daily measurements and to keep accurate records. Communicating your findings to your doctor can be a valuable aid in diagnosing and preventing potential health problem.*

In 2013, the European Society of Hypertension defined a blood pressure classification shown in the table below. Nevertheless, this classification is just a general guide, as blood pressure varies from one person to another, based on age, weight, and health.

IT IS RECOMMENDED THAT YOU CONSULT WITH YOUR PHYSICIAN TO DETERMINE YOUR NORMAL

## BLOOD PRESSURE.

This blood pressure chart reflects categories defined by the European Society of Hypertension.

BLOOD PRESSURE CLASSIFICATION CHART (IN MM HG)				
Category	Systolic (upper)		Diastolic (lower)	WHO indication in colour
<b>Optimal</b>	less than 120	and	less than 80	green
<b>Normal</b>	between 120 and 129	and/or	between 80 and 84	green
<b>Normal high</b>	between 130 and 139	and/or	between 85 and 89	green
<b>Stage 1 - Mild Hypertension</b>	between 140 and 159	and/or	between 90 and 99	yellow
<b>Stage 2 - Moderate Hypertension</b>	between 160 and 179	and/or	between 100 and 109	orange
<b>Stage 3 - Severe Hypertension</b>	greater than or equal to 180	and/or	greater than or equal to 110	red

*Note: There is no universally accepted definition of hypotension (low blood pressure), but systolic pressure below 99 mm Hg is usually regarded as hypotension.*


### 3. PRECAUTIONS BEFORE USE

- Do not confuse self-monitoring with self-diagnosis. Blood pressure readings should be interpreted by a physician or trained healthcare professional, who is familiar with your medical history. Measurements do not replace your doctor's treatment or regular check-ups.
- If you are taking medication, consult with your physician to determine the most appropriate time to measure your blood pressure.
- Do not use this device on children and the elderly without medical advice.
- This device should not encourage self-medication or adjusting your treatment.
- Do not use the device in a moving vehicle. This can result in erroneous measurements.
- Blood pressure measurements determined with this device are equivalent to those obtained by a trained observer using the cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard for Electronic or automated sphygmomanometers.
- Do not use any other cuff than the one provided.

### 4. FEATURES OF THE TENSIOLASH® KD-795

1. Indication of SYS, DIA, Pulse/Min. measurements and WHO international blood pressure classification level
2. Cardiac arrhythmia detection
3. Automatically stores the 60 last readings.
4. Vocal guidance in 6 languages: French, English, Spanish, German, Arabic, Dutch

### 5. MAIN DEVICE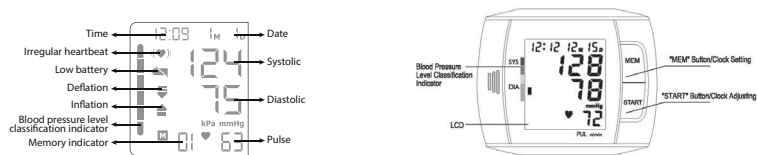


### 6. CHANGING THE BATTERIES

- When the LCD screen displays "Battery", the batteries are low or depleted.
    1. Open the battery cover underneath the device.
    2. Replace the used battery, taking care to observe the indicated polarity. A polarity error may cause damage and void the guarantee. Never insert rechargeable batteries. Use disposable batteries. Do not mix new batteries with used batteries.
    3. Replace the cover.
  - Do not dispose of old batteries with household waste. Take your used batteries to a collection point.
  - If not used for a long period of time, be sure to remove the battery from the device to prevent leakage of liquid that may damage the device.
- Do not use batteries other than the batteries mentioned, do not recharge non-rechargeable batteries, or throw them into a fire.

### 7. TIME AND DATE SETUP

1. Press the *START* and *MEM* button simultaneously. The month flashes.
2. Use the *MEM* button to set the month.
3. Press the *START* button to save your settings and display the day.
4. Use the *MEM* button to set the day.
5. Follow the steps to set the hour and then the minutes.
6. Press *START* to confirm your selection.
7. The date and time must be adjusted at each battery change.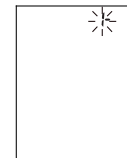

## 8. VOICE SETTING

In *STOP* mode (only time and date displayed on the screen), press the *MEM* button for 3 seconds to change the voice function language.


The screen displays the language and volume (see image opposite):

To change the language, briefly press the *MEM* key (the screen displays L2, and then L3 ... and will give the name of the selected language).

Available in 6 languages: French, English, Spanish, German, Arabic and Dutch


To adjust the volume of a language, press the *MEM* button until the desired volume is reached.

By default, the TensioFlash is in *MUTE* mode.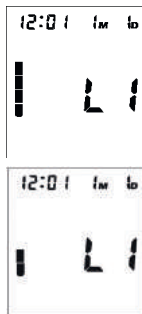

## 9. SOME SUGGESTIONS BEFORE MEASUREMENT

1. Avoid eating, smoking, and exercising for 30 minutes before taking a measurement. Rest for at least 15 minutes before taking a reading.
2. Stress raises blood pressure. Avoid taking measurements when stressed.
3. It is best to take your blood pressure on the left arm.
4. The measurement should be taken in a quiet place, you should be relaxed, in a seated position, your left arm on a table so that the cuff is level with your heart.
5. Sit with your feet flat on the floor (legs uncrossed), place your left arm on the table with your palm facing upward.
6. Remain still and do not talk during the measurement.
7. Record your blood pressure and pulse results in order to show them to your doctor. You should take your blood pressure measurements regularly at the same time of day.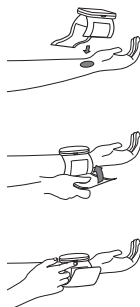


## 10. HOW TO USE THE WRIST CUFF

1. Remove all jewellery (watch, bracelet ...) from your left hand.
2. Palm up, slip on the cuff so that the monitor display faces you in the reading direction.
3. The cuff should be one or two centimetres from your palm.
4. When the cuff is properly positioned, hold it in place by pressing the tab firmly against the Velcro.

Notes:

- Always measure the same arm (normally left) for a better interpretation of the results. If you cannot put the cuff on your left arm, you can use your right arm.

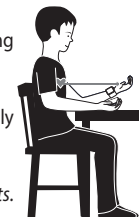## 11. HOW TO MEASURE BLOOD PRESSURE

1. Press the *START* button. All the display characters are shown on the screen.
2. The device displays "zero" on the screen when it is ready to take the measurement. The cuff inflates automatically. When the screen displays the maximum value the pressure is released, the heart beats are detected. The air is automatically evacuated from the cuff.
3. After the measurement is complete, the systolic and diastolic pressure readings and pulse rate are displayed.
4. The result will be automatically stored.
5. Press *START* again to turn the device off. If you forget, the device will stop automatically after about 1 minute.
6. -If there is too much pressure from the cuff, or you feel uncomfortable, please press the *START* button immediately to quickly deflate the cuff.

Notes:

- The value measured will be too low if the cuff is held higher than the heart, or will be too high if the cuff is held lower than the heart.
- The device automatically stores the last 2 sets of 30 readings. If more than 2x30 readings, the earliest data will be deleted.
- During measurement do not speak or move as it may affect the accuracy of the result.

- If you want to stop during the measurement process, just press the **START** button to stop immediately.
- Consult your doctor if in doubt about using the device.

## 12. MEMORY STORAGE AND RECALL

1. Press the **MEM** button to display the stored results. The device displays the number of recorded readings. If no result stored, « - » will be displayed.
2. The screen will display the last measurement. Press the **MEM** button to review data in ascending order. The time and date are stored for all results.
3. To delete measurements from the memory, press the **MEM** button until 0 is displayed on the screen.
4. Press the **START** button to turn off the monitor.

## 13. DEVICE MAINTENANCE

1. Use a soft dry cloth to clean the device. If necessary, use a cloth lightly dampened with tap water.
2. Do not use alcohol, benzene, thinner or other harsh chemicals to clean the device or cuff.
3. Before using please wash your hands. Do not wash or wet the cuff.
4. Avoid placing the device in contact with direct sunlight, high humidity or dust.
5. Fold the pressure cuff and store the device in its box.

*Note: If the blood pressure monitor is left unused for long periods, remove the batteries from the device (otherwise, the batteries may leak and damage the device)*

## 14. PREVENT A MALFUNCTION

- Do not drop the device, avoid jolts and shocks.
- Never insert anything into the device.
- Do not disassemble the device.
- Do not put the device in water.
- The device may not meet its performance specifications if stored or used outside the temperature and humidity ranges specified in the specifications.
- The device retains its safety and performance characteristics for a minimum of 10,000 measurements or three years of use.

## 15. TECHNICAL SPECIFICATIONS

Product name	TensioFlash - Automatic wrist blood pressure monitor - vocal
Model	KD-795
Measurement method	Oscillometric system
Classification	Class IIa medical device
Indication	Digital LCD display
Measuring range	Cuff pressure: 0~300 mm Hg. Pulse: 30~180 Beats/min
Accuracy	Pressure +/- 3 mm Hg Pulse +/- 5%
Inflation	Inflation by electric pump
Deflation	Pressure released by valve

Memory	Stores 60 last readings.
Power source	2 x 1.5V "AAA" alkaline batteries
Battery life	> 50 readings (once a day, at room temperature: 22°C)
Working Temp. & humidity	5~40°C (40~104°F) <85% RH
Storage Temp. & humidity	-20~55°F (-4~131°F) ≤95% RH
External dimensions	340mm (L)/ 72mm (W)/ 30mm (H)
Weight	Approx. 140g (Device and Batteries)
Accessories	Cuff (wrist circumference 13.5 ~ 19.5 cm). User manual, Case, 2 batteries -1.5V AAA
Voltage	3 Volts DC

The manufacturer reserves the right to modify the product technical specifications without prior notice.

## 16. TROUBLESHOOTING GUIDE

If any of the following problems occur when using your TensioFlash®, please refer to this troubleshooting guide. If the problem persists, please contact our customer service at 0 892 350 334 (0,40€/mn).

PROBLEM	POSSIBLE CAUSE	SOLUTION
Nothing is displayed when the START/ STOP button is pressed	No battery inserted	Insert batteries
	The batteries are used	Insert new batteries
	The batteries are not placed in the correct polarity direction	Observe battery polarity
Screen displays abnormal results	The cuff position was not correct or it was not properly tightened	Apply the cuff correctly and try again
	The body posture was not correct during the measurement	Review the suggestions before measurement
	You moved during the measurement or you were not sufficiently relaxed.	Do not move during the measurement. Relax for 10 minutes before taking your blood pressure.
Measurement error (the screen displays EE)	Blood pressure measurement impossible	Press the START button in order to restart the measurement. Check the position of the cuff on the wrist. Check the position of the body during the measurement. Do not speak or move during the reading.

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## ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
RF emissions	CISPR 11 Group 1, Class B	Home healthcare environment
Harmonic distortion	IEC 61000-3-2 Class A	
Voltage fluctuations and flicker	IEC 61000-3-3 Compliance	

Table 2 - Enclosure Port

Phenomenon	Basic EMC standard	Immunity test levels
		Home Healthcare Environment
Electrostatic Discharge	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Radiated RF EM field	IEC 61000-4-3	10V/m 80MHz-2.7GHz 80% AM at 1kHz
Proximity fields from RF wireless communications equipment	IEC 61000-4-3	Refer to table 3

Rated power frequency magnetic fields	IEC 61000-4-8	30A/m 50Hz or 60Hz
---------------------------------------	---------------	-----------------------

Table 3 – Proximity fields from RF wireless communications equipment

Test frequency (MHz)	Band (MHz)	Immunity test levels
		Professional healthcare facility environment
385	380-390	Pulse modulation 18Hz, 27V/m
450	430-470	FM, ±5kHz deviation, 1kHz sine, 28V/m
710	704-787	Pulse modulation 217Hz, 9V/m
745		
780		
810	800-960	Pulse modulation 18Hz, 28V/m
870		
930		
1720	1700-1990	Pulse modulation 217Hz, 28V/m
1845		
1970		
2450	2400-2570	Pulse modulation 217Hz, 28V/m

5240	5100-5800	Pulse modulation 217Hz, 9V/m
5500		
5785		

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**EN**: Laboratoires Visiomed® will repair or replace this product free of charge in the case of defective parts or manufacturing defects, in accordance with the conditions mentioned below as follows:

**DURATION**: 24 MONTHS RETURN TO WORKSHOP

**LIMITS AND EXCLUSIONS**: This guarantee concerns only the original final purchaser. A purchase invoice, or another proof of purchase, with this guarantee card will be required to obtain an after-sales service, in accordance with this guarantee. This guarantee card will not be extended to another person only the original final purchaser. This guarantee becomes void if the serial numbers on the product are modified, replaced, illegible, absent, or if repair has been carried out by a service not approved, including the user.

This guarantee covers only the defects of the material or parts, occurring during normal use of the product. It does not cover the damage caused during the transport of the apparatus, causes due to repairs being carried out by the distributor, by any modifications undertaken, any connection of equipment not approved by Laboratoires Visiomed®, or causes contrary to those written in the user manual or notice. Moreover, the present guarantee does not cover damage due to falls, bad handling, bad installations, damage by fire, floods, lightning, or any other natural disaster. This guarantee does not cover the packing of the material, the accessories, the defects caused by commercial exposure of the product, show room, sale space, demonstration etc... Normal maintenance, cleaning and the replacement of parts where wear is normal, are not covered by the terms of this guarantee. Laboratoires Visiomed® and its representatives and agents will not in any case be held responsible for any damage and consecutive damages due to the mishandling of this product. This guarantee is the only valid one at Laboratoires Visiomed®, any other guarantee (commercial guarantee) except this one will not be taken into account.

**IMPORTANT**: During the guarantee period if you are dissatisfied with the repairs of this product, please contact the Laboratoires Visiomed® customer service.

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**EN :** Disposal of Waste Equipment by Users in Private Households in the European Union.

For more information about where you can drop off your waste equipment for recycling, please contact your local city office or your household waste disposal service.

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**EN :** TensioFlash is a brand registered by Laboratoires Visiomed. The constructor keeps the right to modify the photographs and characteristics without previous notice. Registered design. The logos and trademarks are the property of their respective owners.

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© 02/2019 KD-795